LANSINOH®

- **Lansinoh Breast Pump Instructions**

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LANSINOH®

Contents

Important Safeguards	1	Using the Collection Kit	6
Pump Diagram	2	Using the Lansinoh Electric Breast Pump	6
Cleaning Instructions	3	Troubleshooting	8
Pumping Tips	4	Technical Data	8
Guidelines for Milk Storage and Thawing	4	Replacement Parts	9
About the Collection Kit	5	Warranty	10

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be maintained, including the following:

READ ALL INSTRUCTIONS BEFORE USING

Danger: To reduce the risk of electrocution:




- ▶ Always unplug this product immediately after using.
- ▶ Do not use while bathing.
- ▶ Do not place or store this product where it can fall or be pulled into a tub or sink.
- ▶ Do not place in or drop pump into water or other liquid.
- ▶ Do not reach for an electrical product that has fallen into water. Unplug immediately.

Warning: To reduce the risk of burns, electrocution, fire, or injury to persons:

- ▶ This product should never be left unattended when plugged in.
- ▶ Close supervision is necessary when this product is used near children or invalids.
- ▶ Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- ▶ Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- ▶ Keep the cord away from heated surfaces.
- ▶ Never use while sleeping or drowsy.
- ▶ Never drop or insert any object into any opening or tubing.
- ▶ Do not use outdoors with the AC adapter or operate where aerosol spray products are being used or where oxygen is being administered.

SAVE THESE INSTRUCTIONS

Lansinoh Breast Pump

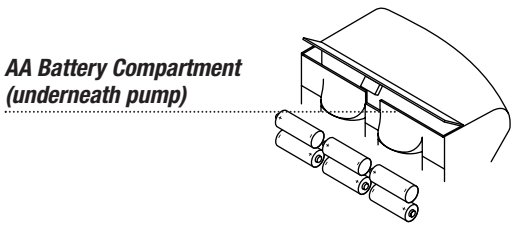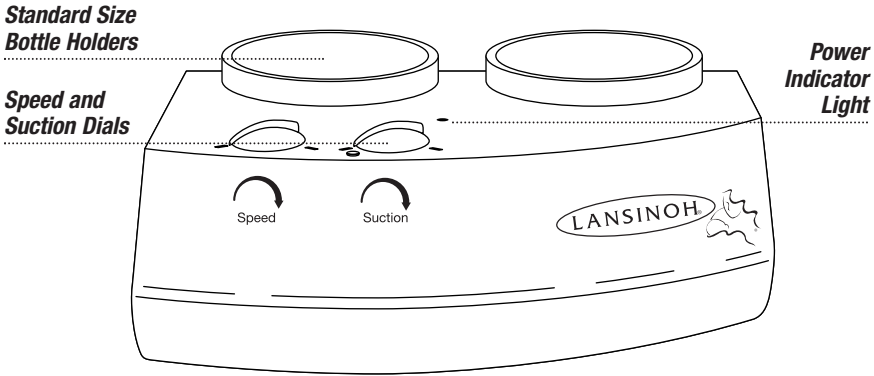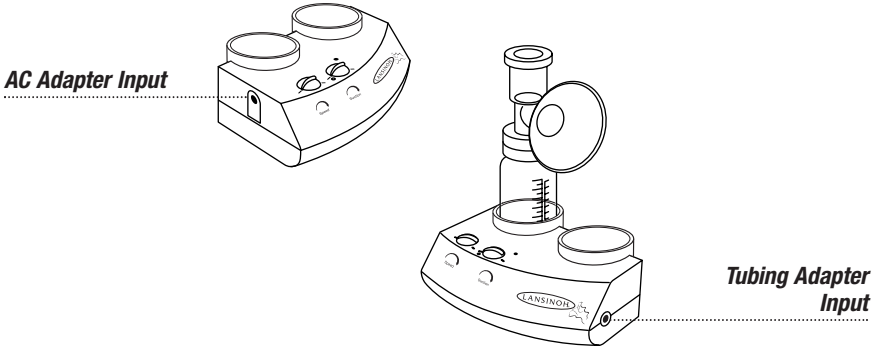For Your Health and Safety

The Lansinoh breast pump is a personal care item and for hygienic reasons should never be shared between mothers or resold from one mother to another. If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breastmilk using the Lansinoh Double Electric Breast Pump will not reduce or remove the risk of transmitting the virus to your baby through your breastmilk.

Before Each Use...

- ▶ Always wash hands thoroughly with soap and water before handling the collection kit, the pump, or other attachments.
- ▶ Take the collection kit apart to clean.

Cleaning Instructions

Unless otherwise instructed by your healthcare provider, follow these sanitizing and cleaning instructions.

Sanitization

Prior to using the kit for the first time, boil all parts of the collection kit for 20 minutes - except for the tubing, white cap and white pump connector. Remove all parts from water immediately after boiling. Completely air dry before assembly and next use.

Cleaning

Set aside the tubing, white cap and white pump connector. These items do not need to be washed or sanitized because the silicone diaphragm prevents them from coming into contact with the breast milk.

The breast flange and bottles may be cleaned in the upper compartment of your dishwasher or hand washed with dish soap in hot water. For best results, we recommend using a baby bottle brush to clean the bottle and a baby nipple brush to clean the insides of the breast shield.


DO NOT use solvents or abrasives. Rinse with hot, clean water. All other parts of the collection kit should be hand washed to prevent loss and extend their usefulness.

Remove the white valves, breast flanges, silicone diaphragms, breast shields and collection bottles for cleaning after each use of the pump. The silicone diaphragms and white valves should be hand washed to prevent loss and extend usefulness. It is important to properly reassemble these parts for the collection kit to operate as designed with the Lansinoh double electric breast pump.

Completely air dry cleaned parts on a clean towel or drying rack. If they will not be used for several hours, cover them with a clean towel.

NOTE: Use caution when cleaning the small white valve. It is critical for proper suction. Do not use excessive force and do not clean with a baby bottle brush or a nipple brush. Gently wash by hand in hot soapy water and rinse under hot running water.

NOTE: Before using the Milk Collection Kit, visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. If discovered, they may affect the functionality of the collection kit and should be discarded. Please visit www.lansinoh.com or call 1-800-292-4794 for further guidance and replacement parts.

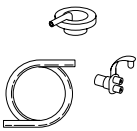Do not wash the tubing, white cap and white pump connector.

Tips about Pumping

Preparing to Pump

The most important part of successful pumping is the let-down reflex. This is when your milk begins to flow freely. Warm wet compresses applied to the breasts and light, circular massaging of the breasts prior to and/or during a pumping session will help this natural reflex occur. Thinking of or looking at your baby, or a photo of your baby, is also helpful.

How Long to Pump

Dual pumping, compared to single pumping, will reduce the time you spend pumping. Overall, a pumping session will usually last about 20 minutes. Remember, successful pumping is a learned art. Early practice sessions may be shorter or longer than stated here and may result in only a small amount of milk collected. Milk is produced on a supply and demand basis. When pumping full-time for a baby unable to nurse at the breast, emptying the breasts regularly stimulates and maintains the milk supply. If breastfeeding problems occur, call your physician, an international board certified lactation consultant, or a La Leche League Leader.

Pumping Options

- ▶ You may pump one breast while nursing the baby on the other. This allows you to maximize pumping effectiveness using the let-down reflex stimulated by the baby's sucking.
- ▶ You may pump into any standard sized screw-top baby bottle.
- ▶ Make sure that the nipple tunnel of the collection kit is tilted slightly downward to allow the milk to flow naturally toward the bottle.

Guidelines for Milk Storage and Thawing.

Milk Storage

Fresh breast milk should be stored in a clean, sealed container. If kept at room temperature (66° to 72°F) it should be used within ten (10) hours. Use refrigerated (32° to 39°F) breast milk within eight (8) days; then freeze or discard. A refrigerator style freezer (temperatures vary), such as a side-by-side or top/bottom, can keep milk safely stored for three to four (3 to 4) months. In commercial deep freezers (0°F or -20° C), milk can be safely stored for six (6) months or longer.

DO NOT store milk in freezer door. The door tends to be the warmest part of the freezer.

Date all milk and use the oldest milk first.

Thawing

Thaw frozen milk in the refrigerator or place the container under cool running water, gradually warming the water until the milk is thawed. Thawed milk can be kept in a refrigerator for 24 hours. **DO NOT** refreeze milk once it is thawed.

Gently shake container of thawed milk before feeding to baby to mix the layers that have separated. Breast milk is not homogenized and the "cream" will rise to the top of the bottle. Breast milk can vary in color and consistency depending upon the time of the day that the milk was expressed, what you have eaten, and the age of the baby at the time of pumping.

DO NOT MICROWAVE HUMAN MILK! Microwaving alters the composition of the milk. Microwaving also may create "hot spots" in the milk that could burn the infant.


DO NOT thaw breast milk in hot or boiling water.

Breast Milk Storage Guide

	<i>Deep Freeze</i> <i>0°F or Less</i>	<i>Refrigerator</i> <i>Style Freezer</i> <i>Self-contained</i> <i>Freezer</i> <i>Approx. 0°F</i>	<i>Refrigerator</i> <i>32°F to 39°F</i>	<i>Room</i> <i>Temperature</i> <i>66°F to 72°F</i>
Fresh Breast Milk	6+ Months	3 – 4 Months	8 Days	10 Hours
Thawed Breast Milk	Do Not Refreeze	Do Not Refreeze	24 Hours	1 Hour


Source: La Leche League International

About the Collection Kit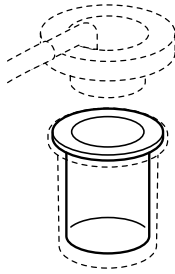

The patented diaphragm system of the collection kit prevents milk from backing up into the tubing and the pump motor.

Using the Collection Kit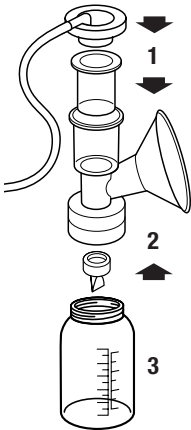



Your collection kit comes preassembled in Lansinoh product packaging.

Clean collection kit parts (except tubing, white cap and white pump connector) as described in the “Cleaning Instructions” section (see page 3). Once all parts are clean and dry, assemble the collection kit as follows:

- 1 Place diaphragm into piston chamber and snap white cap onto piston chamber.
- 2 Gently secure white valve into lower portion of breast flange.
NOTE: Use caution when handling and cleaning the small white valve (see page 3).
- 3 Screw collection bottle onto breast flange.
- 4 Insert white pump connector, at end of tubing, into tubing adapter input on the pump.

Hold the collection kit to your breast(s) so that the nipple is aligned to extend straight toward the nipple tunnel of the breast flange. Make sure the breast completely fills the breast flange so that a vacuum is formed and no air escapes. The collection kit and nipple tunnel must tilt slightly downward to allow milk to flow naturally toward the bottle.

- 5 To dual pump, you may use both hands to hold the collection kit against each breast. Alternatively, you may use one hand to hold one breast flange against the breast and the forearm to hold the other breast flange against the other breast.

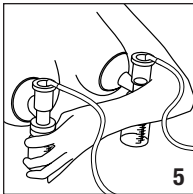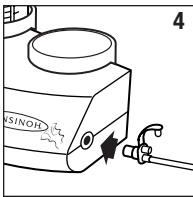
To Single Pump:
Remove one tube and close the white pump connector.

To Dual Pump:
Both tubes should be attached to the white pump connector.

You are now ready to pump!

Using the Lansinoh Double Electric Breast Pump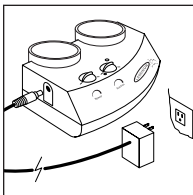

Read the section titled “Using the Collection Kit” before continuing.


Power Options

The pump may be operated under any of the following power supply modes:

AC Adapter (included)

Connect the AC adapter to the power receptacle located on the left side of the pump. Plug the AC adapter into an available electrical outlet.

Electrical safety can only be achieved when the breast pump is connected to the AC Power Adapter supplied with the product. (See “Technical Data” section, page 8.)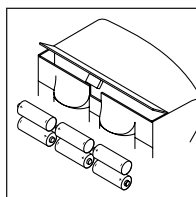

6 AA Alkaline or Rechargeable Batteries (not included)

Open the battery cover under the pump and insert 6 AA batteries. Make sure the polarity (+) and (-) is correct as indicated. Close the compartment by gently pressing the cover until it snaps into place.


Remove batteries if pump is not in use for a long period of time. Dispose of batteries according to regulations for environmental protection and recycling.

If batteries are in pump, but AC adapter is in use, the pump will use the adapter to power the pump, not the batteries.

For the most economical use, we recommend using your pump with the AC adapter provided.

The Adjustable Difference

Your Lansinoh Double Electric Breast Pump allows you to adjust both suction and speed settings. Choose the settings that are most comfortable for you and result in the greatest milk expression.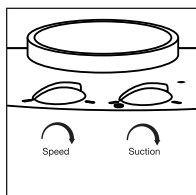

Speed Dial

This controls the speed. Turn the dial clockwise to increase the speed. Some mothers have found that their milk flows more quickly if the pump speed is initially set on "high."

Suction Dial

This controls the suction strength. Start the pump by turning the *Suction dial* clockwise. Continue to turn the suction dial clockwise to increase the suction to the highest level *comfortable* for you.

Ready to Pump

- 1 Refer to sections titled "Using the Collection Kit" and "Pumping Tips."
- 2 Set up Collection Kit for single or double pumping. Turn the *Suction* control dial clockwise to switch on the pump. (See previous sections entitled "Speed Dial" and "Suction Dial.")

NOTE: *If the suction on your breast reaches uncomfortable levels, reduce the suction level with the Suction control dial, or disconnect the white pump connector from its receptacle on the electric pump.*

- 3 Start the *Speed* dial at the highest setting comfortable for you. Find the speed that most closely corresponds to the sucking rhythm of your baby. The speed setting may be adjusted from 30 to 60 cycles per minute to accommodate individual comfort levels.

NOTE: *Some mothers find it helpful to begin pumping on a high setting, then lower the setting once milk starts to flow.*

- 4 To clean your breast pump, wipe the casing with a damp cloth, using only water or mild soap. To clean and sanitize your collection kit, refer to the section "Cleaning Instructions."
- 5 If you purchased the optional Cooler Pack and Freezer Pack as accessories to your Lansinoh Double Electric Breast Pump, you may safely store up to six (6), 4 oz. bottles in the room at an ambient room temperature of 66°-72° F for up to ten (10) hours. Freezer packs should be removed once thawed and refrozen.

NOTE: *When a pumping session is finished, turn the pump off. Make sure there is no suction on the nipple(s) before removing the breast flanges from the breast. If necessary, break the suction by inserting a finger between the breast flange and the breast. Once the suction is broken, the breast flange can be removed safely.*

Technical Data

Power Requirements

AC Adapter *(included)*

AC Adapter – North American Plug Configuration

9VDC/1.3A, 120V/60Hz, Lansinoh Stock No. 622401, direct plug in AC power adapter.

Batteries *(not included)*

Six AA batteries, 1.5V alkaline or 1.2V rechargeable.

Regulatory Compliance

UL and CSA approved with North American power adapter.

UL File Number E189700

CSA File Number LR765251

Troubleshooting the Lansinoh Double Electric Breast Pump

Interruption of Electrical Service

If you experience an interruption in the normal operation of the electrical and/or mechanical functioning of the pump, simply do the following:

- ▶ Unplug the AC adapter cord from the electrical outlet.
- ▶ Wait a few seconds, then plug the AC adapter cord back into the electrical outlet.


Decreased Suction

If it feels like the suction has decreased, try the following:

- ▶ Remove the tubing and white connector from the side of the pump and place a finger over the pump outlet. If you feel suction, the pump is operating properly, but the collection kit may not be correctly assembled or the white valve may be damaged. If no suction is felt on your finger, try unplugging the adapter from the electrical outlet, wait a few seconds, and plug it back into the electrical outlet.
- ▶ Make sure the collection kit has been properly assembled. Check the assembly information in the section "Using the Collection Kit", page 6.
- ▶ Inspect the white valve for a tear or pinhole. This component is critical to achieving proper suction. Replace the white valve with one of the spare valves provided with the product.
- ▶ Check the tubing adapter on the side of the pump to make sure it is closed when single pumping. If open, close the adapter.

Replacement Parts

Pump Body and Accessories

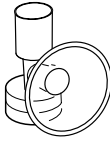621262 Breast Flange

17211 Areola Stimulator

402326 Valve

620551 Locking Ring

623954 Locking Disk

844331 4-ounce Polycarbonate Bottle


623129

Adapter Cap

402332

Silicone Diaphragm

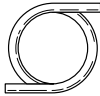
402333 Tubing

620559 White pump connector

617063 Freezer Pack
1 each

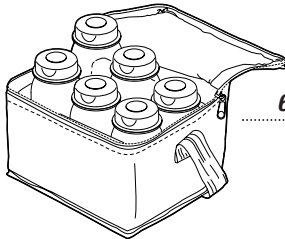623327 cooler

Limited Consumer Product Warranty (United States)

This product is warranted to the original consumer purchaser (the “purchaser”) to be free from defects in material and workmanship which are not commercially acceptable for the period of one year from the date of purchase on the pump mechanism and the period of 90 days on all other product components. Warranty coverage terminates if you sell or otherwise transfer this product to another person. This warranty gives you specific legal rights and you may also have other rights, which vary, by location.

LANSINOH MAKES NO EXPRESS WARRANTY OF ANY KIND REGARDING THE PRODUCT OTHER THAN THOSE WARRANTIES EXPLICITLY SET FORTH HEREIN. ANY IMPLIED WARRANTY, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR ANY IMPLIED WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE, TO THE EXTENT PERMITTED BY LAW, SHALL BE LIMITED IN DURATION TO A PERIOD OF 90 DAYS FROM THE DATE OF PURCHASE BY THE ORIGINAL PURCHASER.

In the event the product is found by Lansinoh to not meet the above limited warranty, as purchaser’s sole and exclusive remedy Lansinoh will repair or, at the option of Lansinoh, replace this product without charge for such replacement or parts or labor. The purchaser shall bear all expense for returning this product to Lansinoh. This warranty does not apply to any product that has been subjected to misuse, abuse, or alteration. Only Lansinoh brand replacement parts are recommended for use with Lansinoh breastfeeding products. Improper or incorrectly performed maintenance or repair, or use of non-Lansinoh brand replacement parts may invalidate this warranty.

TO THE EXTENT PERMITTED BY LAW, LANSINOH SHALL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES INCLUDING, BUT NOT LIMITED TO, REPLACEMENT COSTS RESULTING FROM THE BREACH OF ANY WRITTEN OR IMPLIED WARRANTY.

If you wish to make a claim under this warranty, you must return the product to Lansinoh Laboratories Inc., prepaid, together with your dated bill of sale or other dated proof of purchase and a brief statement of the problem, along with a Return Authorization (RA) number. You may obtain the appropriate Return Authorization number and shipping address by calling the phone number or sending an email to the email address provided below.

To obtain address to Return goods to: 1-800-292-4794
customerservice@ lansinoh.com

To obtain a Return Authorization Number: 1-800-292-4794
customerservice@ lansinoh.com

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©2005 Lansinoh Laboratories, Inc. All rights reserved.
Lansinoh Laboratories, Inc
Lansinoh breastfeeding products are manufactured and distributed by Lansinoh Laboratories, Inc in the US.